NanoBone®

■ Knochenaufbau in neuer Dimension

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- Canullo L, Patacchia O, Sisti A, Heinemann F: Implant Restoration 3 months after One Stage Sinus Lift Surgery in Severely Resorbed Maxillae: 2-Year Results of a Multicenter Prospective Clinical Study, *Clin Implant Dent Relat Res* 2010, in press
- Heinemann F, Mundt T, Biffar R, Gedrange T, Götz W: A 3-year clinical and radiographic study of implants placed simultaneously with maxillary sinus floor augmentations using a new nanocrystalline hydroxyapatite, *J Physiol Pharmacol* 2009, in press
- Klein MO, Gotz H, Duschner H, Wagner W: Knöcherne Integration eines alloplastischen Knochenersatzmaterials (NanoBone®) im Sinuslift; Bony integration of an alloplastic bone substitute material (NanoBone®) after maxillary sinus augmentation, *Z Zahnärztl Impl* 2009; 25 (4):20-28
- Mertens C, Steveling H: Use of Synthetic Bone Blocks as an Alternative to Autologous Bone Block Grafts, *Implants, International Magazine of Oral Implantology* 2009; 4:30-32
- Meier J: Experiences with the nanostructured bone substitute NanoBone™ in particular and block form: Prospective histological and clinical trial with 3 years follow-up, *EAO 18th International Meeting Monaco 2009*, Poster 245
- Xu W, Holzhüter G, Sorg H, Wolter D, Lenz S, Gerber T, Vollmar B: Early matrix change of a nanostructured bone grafting substitute in the rat., *J Biomed Mater Res B Appl Biomater.* 2009 Nov; 91(2):692-9
- Reichert C, Al-Nawas B, Smeets R, Kasaj A, Götz W, Klein MO: In vitro proliferation of human osteogenic cells in presence of different commercial bone substitute materials combined with enamel matrix derivatives, *Head & Face Medicine* 2009, 5:23
- Punke C, Zehlicke T, Boltze C, Pau HW: Investigation of a new highly porous hydroxyapatite matrix for obliterating open mastoid cavities - application in guinea pigs bulla, *Laryngorhinootologie* 2009 Apr; 88(4):241-6
- Canullo L, Voza I, Caricato F, Dellavia C: Maxillary sinus floor augmentation using a nano-crystalline hydroxyapatite silica gel, A prospective study—Histological results after 3 months of healing Implants, *International Magazine of Oral Implantology* 2009; 2:24-27
- Stübinger S, Ghanaati S, Orth C, Hilbig U, Saldamli B, Biesterfeld S, Kirkpatrick CJ, Sader RA: Maxillary sinus grafting with a nano-structured biomaterial: preliminary clinical and histological results., *Eur Surg Res.* 2009;42(3):143-9. Epub 2009 Jan 29.
- Canullo L, Dellavia C: Sinus lift using a nano-crystalline hydroxyapatite silica gel in severely resorbed maxillae: histological preliminary study, *Clin Implant Dent Relat Res* 2009 Oct; 11 Suppl 1:e7-13. Epub 2009 Feb 13
- Abshagen K, Schrodi I, Gerber T, Vollmar B: In vivo analysis of biocompatibility and vascularization of the synthetic bone grafting substitute NanoBone® *J Biomed Mater Res A* 2009 Nov; 91(2):557-66
- Kruse A, Jung RE, Nicholls F, Zwahlen RA, Hämmerle CHF, Grätz KW, Weber FE: Comparison of synthetic HA/SiO2 matrix and bovine derived HA, *EAO 17th International Meeting Warsaw 2008*, Poster 243
- Lenz S, Kirchhoff M, Gerber T: Enhanced osseointegration of implants with a nanostructured bioactive coating, *EAO 17th International Meeting Warsaw 2008*, Poster 391
- Meier J, Wolf E, Bienengraber V: Einsatz des synthetischen nanostrukturierten Knochenaufbaumaterials NanoBone® bei Sinusbodenelevation, *Implantologie* 2008;16(3):301-314
- Götz W, Gerber T, Michel B, Lossdörfer S, Henkel KO, Heinemann F: Immunohistochemical characterization of nanocrystalline hydroxyapatite silica gel (NanoBone®) osteogenesis: A study on biopsies from human jaws *Clin Oral Impl Res* 2008; 19:1016-26
- Punke C, Zehlicke T, Boltze C, Pau H-W: Experimental Studies on a New Highly Porous Hydroxyapatite Matrix for Obliterating Open Mastoid Cavities, *Otol Neurotol* 2008 Sep; 29(6):807-11
- Harms C, Helms K, Taschner T, Stratos I, Gerber T, Lenz S, Vollmar B, Mittlmeier T: Histomorphometric and micro-CT analysis of the osteoneogenic capacity in the metaphysis of the sheep after implantation of nanocrystalline bone grafting substitute NanoBone®, *Chirurgisches Forum* 2008, Band 37; 253:255
- Schrodi I, Abshagen K, Gerber T, Vollmar B: In vivo analysis of biocompatibility and vascularization of the synthetic bone grafting substitute NanoBone®, *Chirurgisches Forum* 2008, Band 37; 251:252
- Kasaj A, Willershausen B, Reichert C, Gortan-Kasaj A, Zafiroopoulos GG, Schmidt M: Human periodontal fibroblast response to a nanostructured hydroxyapatite bone replacement graft in vitro, *Archives of Oral Biology* 2008; 53:683-689
- Stübinger S, Ghanaati SM, Orth C, Booms P, Kirkpatrick C, Sader R: A new nano-structured and synthetic biomaterial promotes reconstruction of alveolar ridge defects after dental trauma: A preliminary report of clinical and animal studies, *Poster: IADT 2008*

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ARTOSS GmbH